

### Use of Rodent Multi-Chamber Anesthesia Units:

**Clean** with Oxivir spray/wipes before & after use.

**O<sub>2</sub> gauge** is above **100 PSI** and Vaporizer isoflurane fluid level is full.

Charcoal WAGS canister is **<50g of original weight**.

Confirm "Bains" **breathing circuit** tubing/nose cone are in good working order.

**Induction** at **2-3%** isoflurane vaporizer setting.

**Maintenance** at **1-2%** isoflurane vaporizer setting.

**Oxygen flow rates** at **0.5 L/minute** for each chamber or Bains circuit in use.

# of Chambers/Circuit	O <sub>2</sub> Flow Rate (L/min)
1	0.5
2	1.0
3	1.5
4	2.0
5	2.5
6	3.0

SC #288  
Eff. 9/23